


Easy to use:


Step by step inhalation with the Easyhaler®

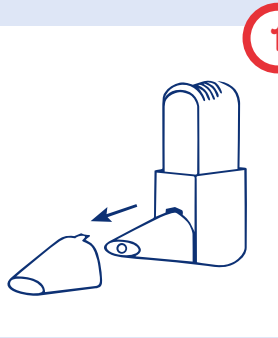

1

- Remove the safety cap from the mouthpiece


Shake

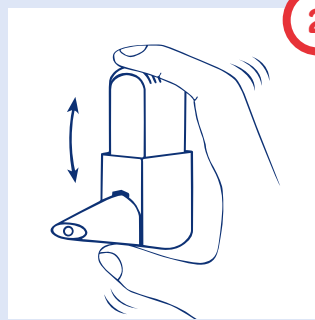

2

- Shake the Easyhaler® vigorously 3 to 5 times


Click

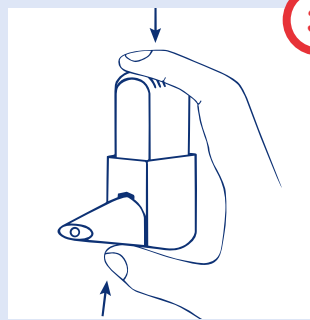

3

- Hold the inhaler upright
- Squeeze firmly between your thumb at the base and your index finger at the top, until you hear a click
- Then let it click back to its original position

Inhale

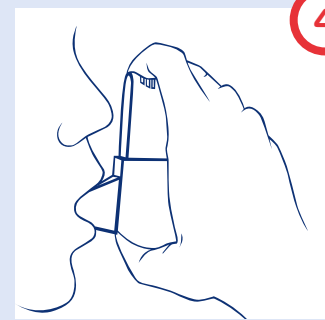


4

- Breathe out normally (but NOT into the Easyhaler®)
- Close your lips tightly around the Easyhaler®
- Breathe steadily and deeply through the Easyhaler®
- Remove the inhaler and hold your breath for 5 seconds
- Replace the safety cap onto the mouthpiece

If you are using a moisture enclosure

You do NOT need to take the Easyhaler® out of the moisture enclosure for inhalation.

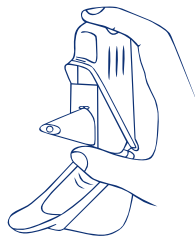

- Open the moisture enclosure
- Remove the safety cap from the mouthpiece
- Inhale, as explained in steps 2 to 4
- Replace the safety cap onto the mouthpiece
- Close the moisture enclosure


Troubleshooting

If...

- you accidentally load the Easyhaler®
- you press the Easyhaler® more than once
- you blow into the inhaler

Then...

- tap the mouthpiece of the Easyhaler® onto a table or your hand

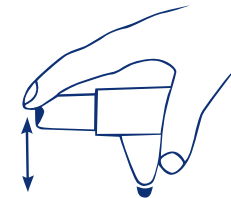


If you have been prescribed more than one dose per application, repeat steps 2 to 4.

! To reduce the risk of a fungal infection, rinse your mouth and gargle with water but DO NOT swallow.

The counter

The Easyhaler® has a counter showing how many single doses are left in the inhaler.

- The counter moves after every 5 applications
- When the numbers turn red: there are only 20 single doses left
- When the counter reaches 0, the Easyhaler® has to be replaced

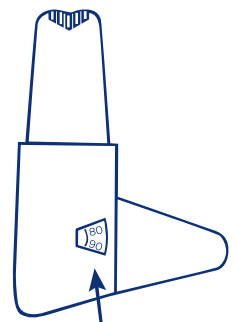

For further information and to watch a video on how to use the Easyhaler, please go to www.easyhaler.de

 **Easyhaler®**

Simple, precise, reliable – with the patient in mind.

Orion Pharma GmbH Notkestraße 9 D-22607 Hamburg
Tel +49 (0) 40 89 96 89-0 Fax +49 (0) 40 89 96 89-96
info.hamburg@orionpharma.com
www.easyhaler.de www.orionpharma.de

 **ORION PHARMA**